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Instructions for use for SCHWEIZER magnifiers

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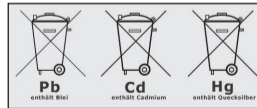
EN Congratulations

With your purchase of a magnifier from SCHWEIZER, you have chosen a top-quality medical product manufactured in Germany in compliance with the latest standards and regulations.

Before using your magnifier for the first time, please read these instructions carefully and observe the safety, cleaning and care information.

Intended use

This device is intended for everyday use, for reading printed matter or magnifying objects.

Cleaning and care

The lens is made from optically clear Plexiglas®. Please use a damp cloth for cleaning, ideally a damp microfibre cloth. Never use solvents such as alcohol, benzene or other cleaning agents for cleaning lightweight lenses (polymer) since they can damage the lens. Small lines that look like hairline cracks at the edges of the lightweight lenses or polymer frame are flow lines that occur during the production process. They do not impair the function or durability in any way.

Safety instructions

Fire hazard: Never leave your magnifier uncovered exposed to the sun. The intensifying effect of the lens can start a fire.

Risk of injury:

- Never use the magnifier to look at the sun.
- Do not look directly at the light source of illuminated magnifiers.
- Avoid mechanical impact or shock such as banging or dropping your magnifier, and ensure it is not exposed to moisture or high temperatures.

Inform other people about these risks, especially children.

Do not dispose of used batteries in household waste.

Please ensure proper disposal (see page 11).

EN Power supply

The ERGO-Lux i mobil series uses a built-in lithium-polymer battery. The ÖKOLUX plus mobil and ERGO-Lux MP series use three standard AAA alkaline batteries (1.5 V, LR3, AM4). The ÖKOLUX plus and MODULAR LED series as well as the TWIN-Lux MP use three standard AA alkaline batteries (1.5 V, LR6, AM3).

The LED light-emitting diodes are subject to the most stringent selection criteria. They have a service life of more than 50,000 hours of continuous operation. It is therefore hardly ever necessary to replace the LED.

Symbol guide

REF

Item no.

LOT

Batch number

Manufacturer

Fragile, handle with care

Protect from moisture

Consult instructions for use

Disposal information

MD

Medical device

CE marking of conformity

Battery safety instructions

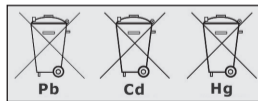
Our batteries are marketed in line with legal requirements. Please read the points below on the safe disposal of our batteries:

- To prevent any harm to health and the environment, used batteries must be disposed of properly at recycling centres or returned to the point of sale. Returning batteries is required by law and free of charge for you.
- Batteries should only be returned in a fully discharged state. The pole terminals of lithium-ion batteries must be insulated with adhesive tape.
- Valuable raw materials such as zinc, iron, nickel and so on can be recovered by recycling used batteries.
- The batteries can be easily removed from our devices for disposal (see the following pages). Magnifiers

containing a built-in rechargeable battery must be disposed of completely.

- The “crossed out wheelie bin” symbol indicates that it is illegal to dispose of used batteries in household waste.

The additional specification of the chemical symbols for lead (Pb), cadmium (Cd) and mercury (Hg) indicates that the content of these materials in the batteries is in legally permissible quantities.

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EN SCHWEIZER – expert knowledge since 1840

At home or on the go, for hobbies, leisure and at work: SCHWEIZER magnification and visual devices for multiple applications improve the quality of everyday life. All glass lenses, illuminated magnifiers, low vision lamps and many other products are made in Germany.

Find out more about Europe's largest and most innovative range of magnifiers, high addition spectacles, telescopic systems, low vision lamps, video magnifiers, blue blocker and comfort filters, special spectacle lenses, fit-over spectacles as well as magnifiers for technical applications such as folding magnifiers, measuring magnifiers, linen testers, watchmaker's loupes and illuminated table top magnifiers at: www.schweizer-optik.de.

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- EN** All rights reserved.
Copyright © 02/2022, A. SCHWEIZER GmbH
A. SCHWEIZER GmbH