# User's Guide

## for Desktop Magnifier *eMag* 240 HD

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Congratulations on purchasing the desktop magnifier **eMag 240 HD**. This electronic device enables you to magnify printed text and objects from 1.9 to 71 times their original size. The **eMag 240 HD** is equipped with a wide screen (format 16:9) with a 61 cm (24") screen diagonal. We hope you enjoy using this high-quality device.

Information in this document is subject to change without notice. No part of this publication may be reproduced or transmitted in any form or any means, electronic or mechanical, for any purpose, without the express written permission of A. SCHWEIZER GmbH.

Copyright © 2015 A. SCHWEIZER GmbH. All Rights Reserved.

# Table of Contents

<b>Transporting and Moving the eMag 240 HD</b> .....	<b>7</b>
<b>Setting Up the eMag 240 HD</b> .....	<b>9</b>
<b>Advanced Video Controls</b> .....	<b>16</b>
Focus Lock and Position Locator Beam .....	16
Turning LED Reading Lights On or Off .....	17
Reading Lines and Reading Masks .....	18
Selecting Lines and Masks .....	19
Opening Configuration Mode and Editing Settings .....	20
Adjusting Width between Lines or Masks .....	21
Using the <b>eMag 240 HD</b> Reading Line .....	22

Continued on next page

# Table of Contents (continued)


<b>Changing the Basic Configuration of the eMag 240 HD</b> .....	<b>23</b>
Settings .....	23
Opening Configuration Mode and Editing Settings .....	23
Customizing Magnification Settings .....	26
Selecting a Magnification Range .....	27
Changing the Zoom Position Display .....	28
Resetting Factory Default .....	30
Displaying the Firmware Version Number .....	31
<b>Connectors on the Back of the Unit</b> .....	<b>32</b>
<b>Important Safety Information</b> .....	<b>34</b>

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
# Transporting and Moving the eMag 240 HD

This image and the information on pages 8 and 9 will show how you can lift, carry and position the eMag 240 HD.

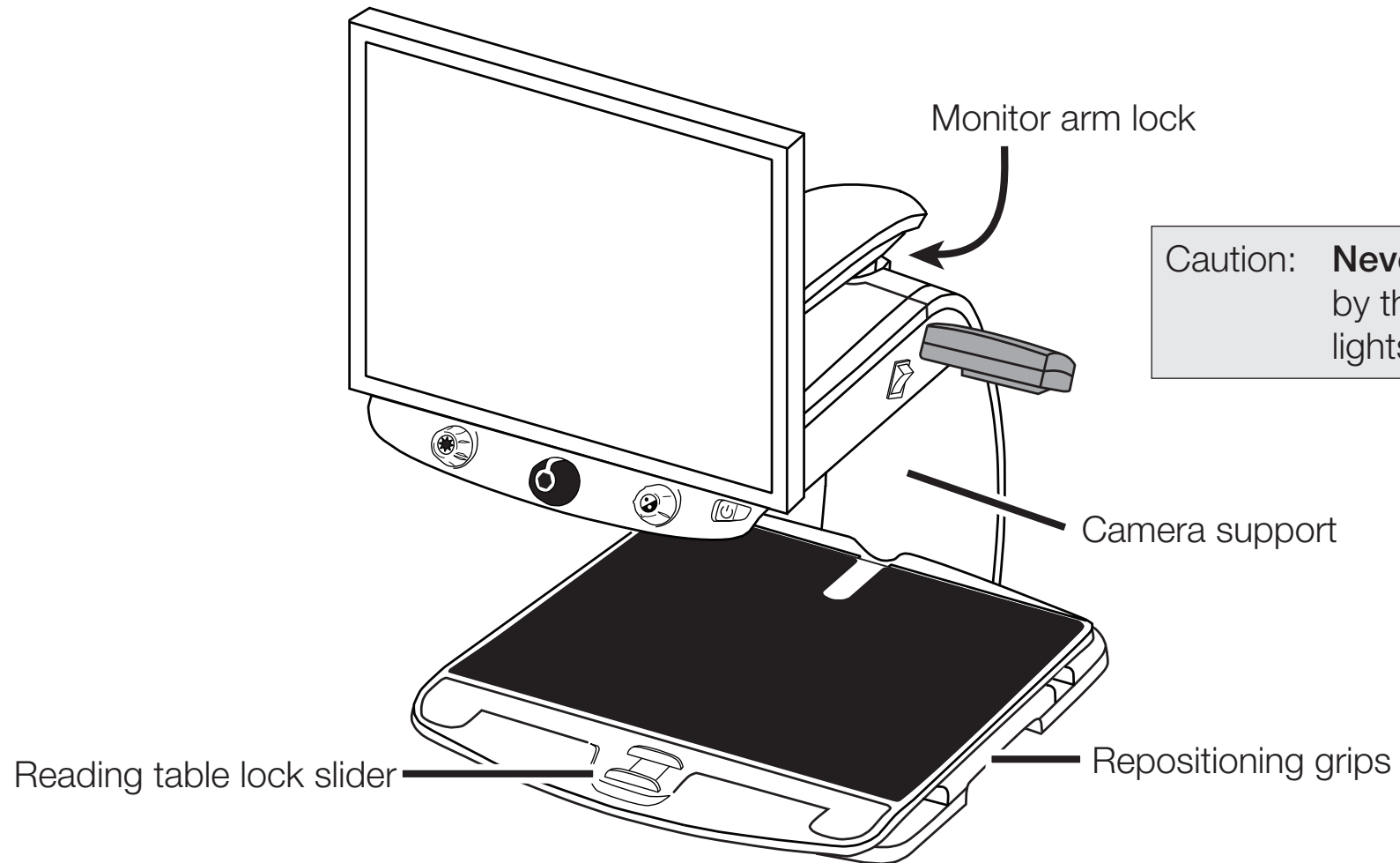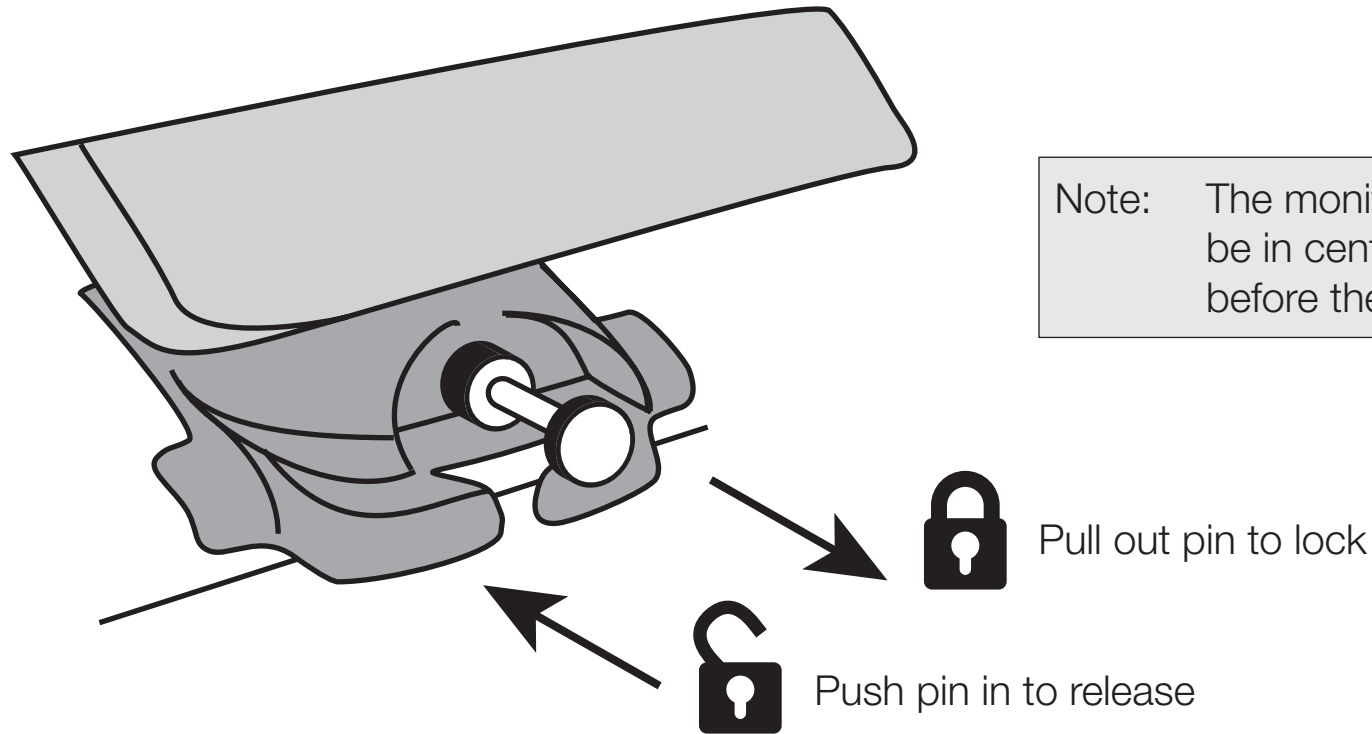
1. Lock the monitor arm to prevent the monitor from moving to the left or right.

2. Lock the reading table in position using the red lock slider.
3. Grip the **eMag 240 HD** by the rear camera support to lift and move it to a stable surface such as a table, desk or worktop. The best way to do this is to wrap your forearm around the camera support from the rear and then lift the **eMag 240 HD**.

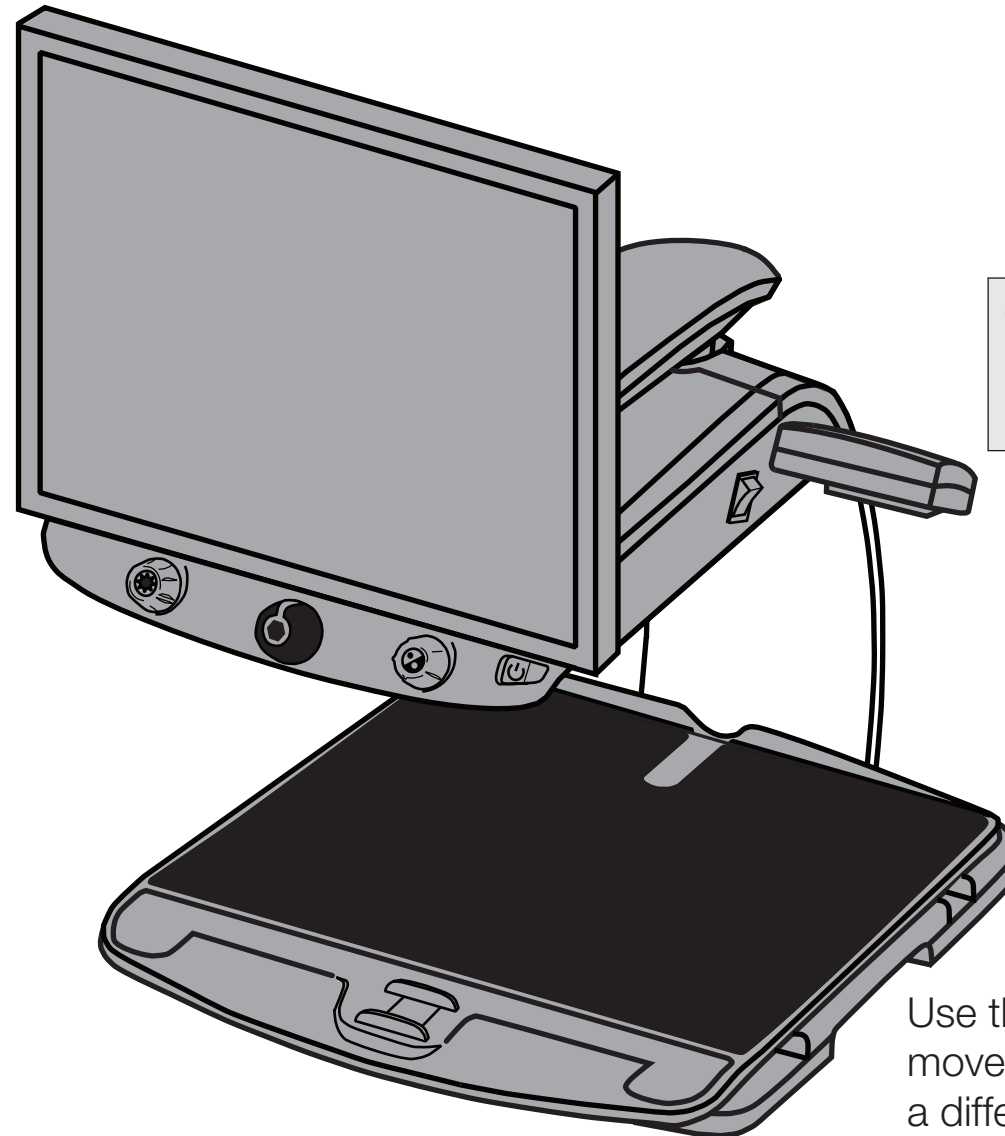Important: Do NOT lift the **eMag 240 HD** by the LED reading lights.

Caution: Never hold by the reading lights!


To lift and transport:  
Reach around the camera support from behind

Use the repositioning grips to move the **eMag 240 HD** to a different location.

4. Once you have placed the **eMag 240 HD** on a stable surface, use the repositioning grips on both sides of the reading table to move and position it to your liking.


# Setting up the eMag 240 HD

This image and the text on pages 11 to 15 show you step by step how to set up the **eMag 240 HD**.

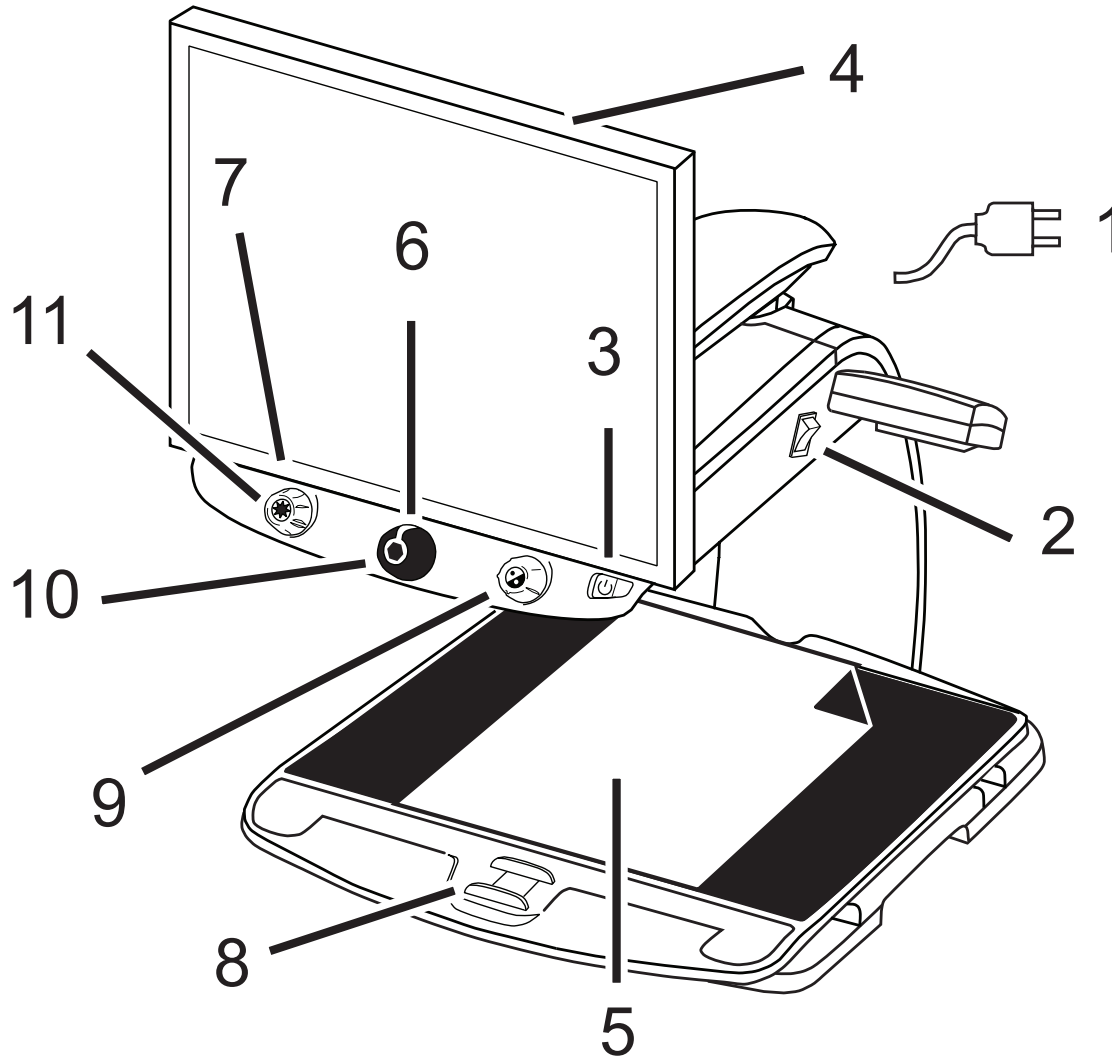- 1 Power cable
- 2 Master power switch
- 3 Power button
- 4 Monitor
- 5 Document on the reading table
- 6 Magnification dial
- 7 Brightness dial
- 8 Reading table lock slider
- 9 Color select dial
- 10 Freeze frame button
- 11 Find button

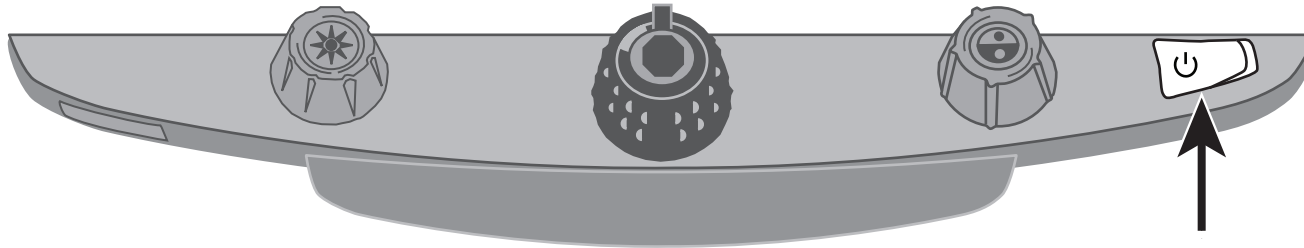1. Connect the power cable: from the middle power connector on the back of the **eMag 240 HD** to a wall outlet.
2. Press the white master power switch near the LED reading light on the right side of the **eMag 240 HD**.
3. Press the green power switch on the right side of the control panel.


Note: If the screen remains blank, check that the monitor is turned on.

4. Hold the sides of the monitor with both hands and move it up or down to change the height, or turn the monitor left or right to adjust the angle for the best view.


If the monitor will not move to the left or right, make sure that the arm lock located at the base of the monitor arm is released (push the locking pin in). Also, make sure that the monitor is raised so that it can move freely from left to right.

For further information on the monitor arm lock, see the section *Transporting and Moving the eMag 240 HD* as already mentioned in this document.

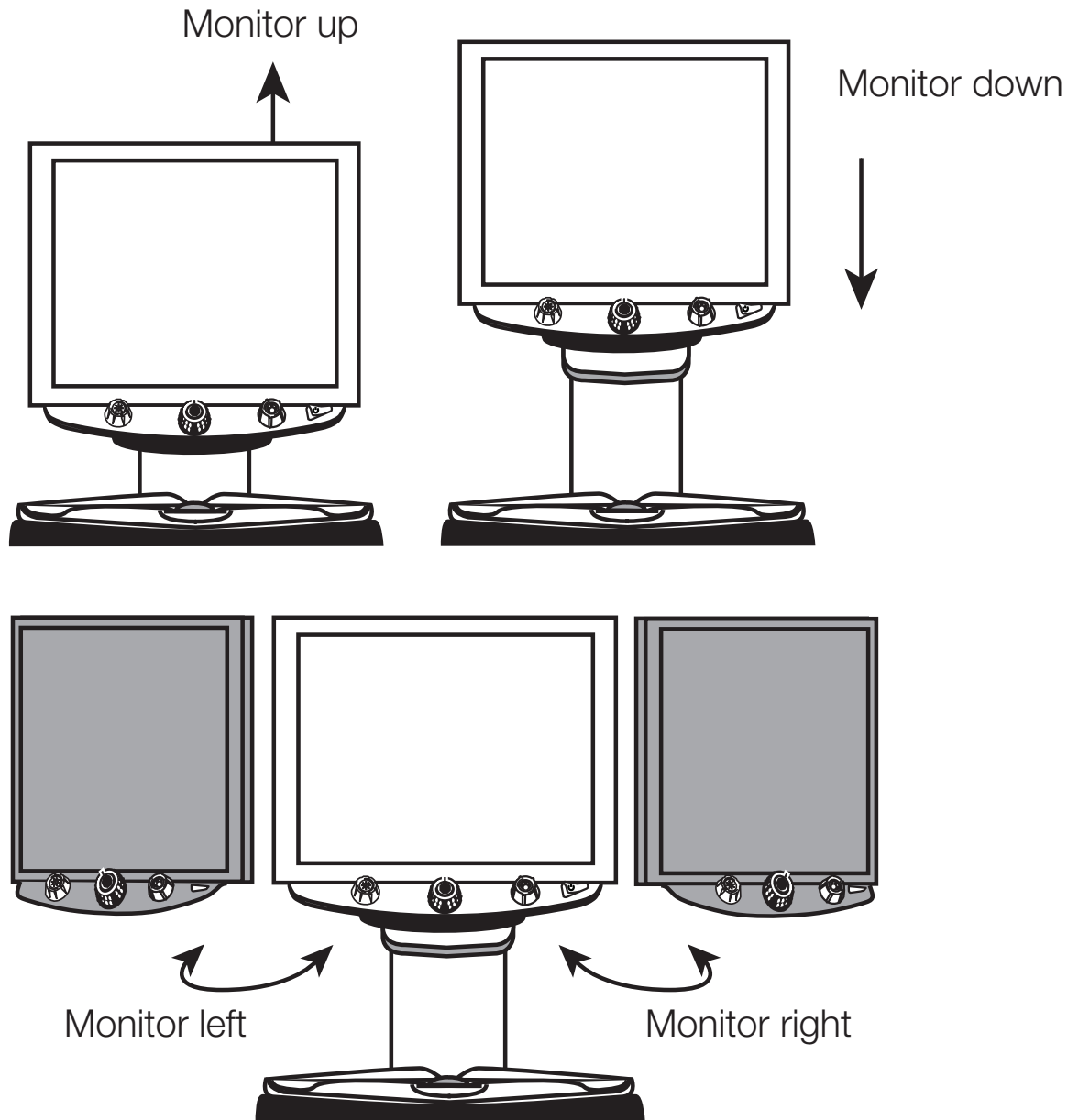
Push the monitor upward and release the monitor arm lock before turning the monitor to the left or right.

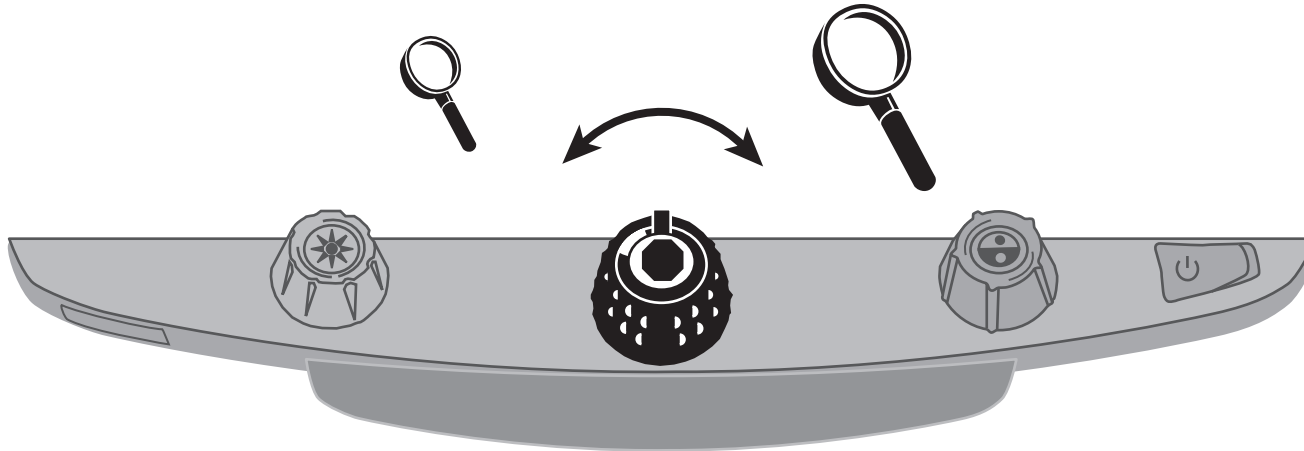
5. Place the document that you want to magnify onto the center of the reading table. Move the document back so that it touches the rear of the reading table. To read the text on a rounded object such as a pill bottle, place it in the indentation in the rear of the reading table. This prevents round objects from rolling on the table.
6. Turn the black magnification dial to adjust magnification.

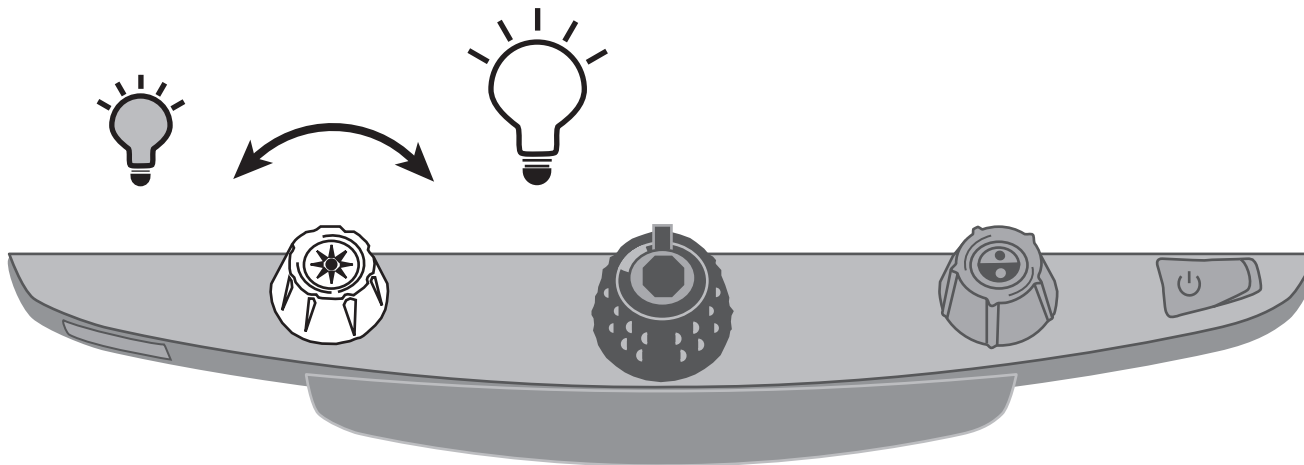
7. Turn the yellow brightness dial to adjust the brightness.

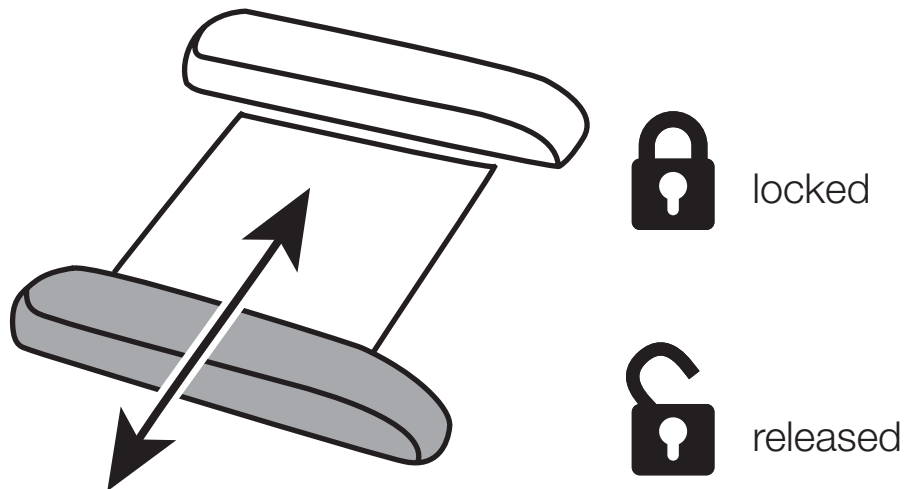
8. Unlock the reading table to move a document so that its text moves back and forth across the screen to make reading easier.

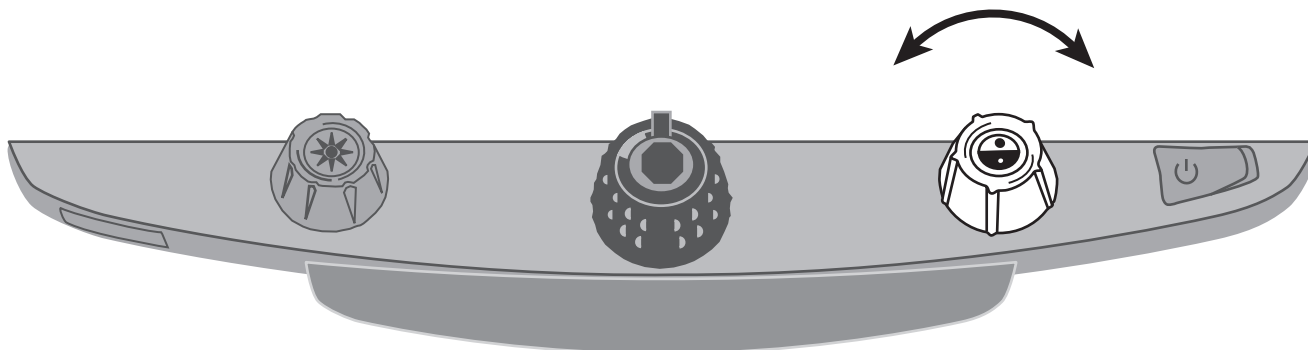
9. Turn the blue color select dial to choose a color combination that best suits your vision needs. The default color settings are: full color, black on white, white on black, yellow on blue and yellow on black.

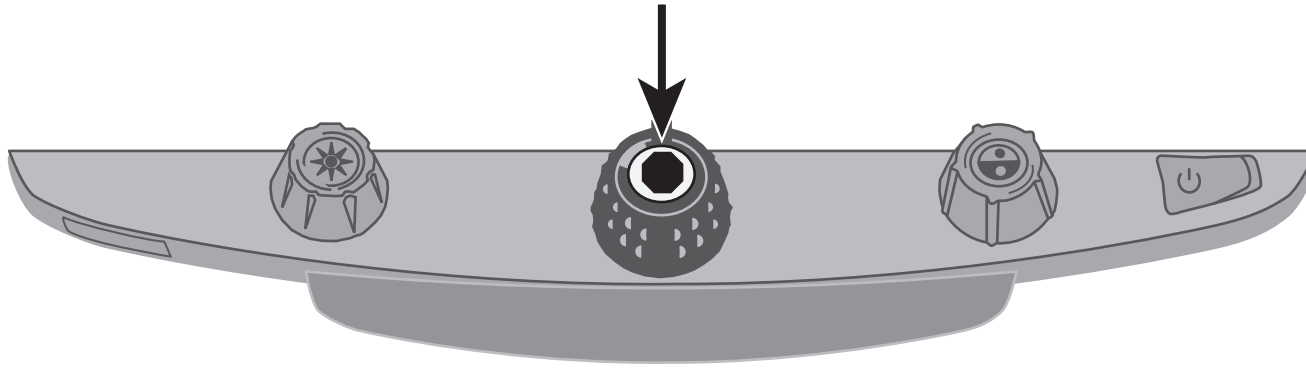10. Press the freeze frame button located in the center of the black magnification dial to freeze the current image on the screen. Press it again to return to the live view.

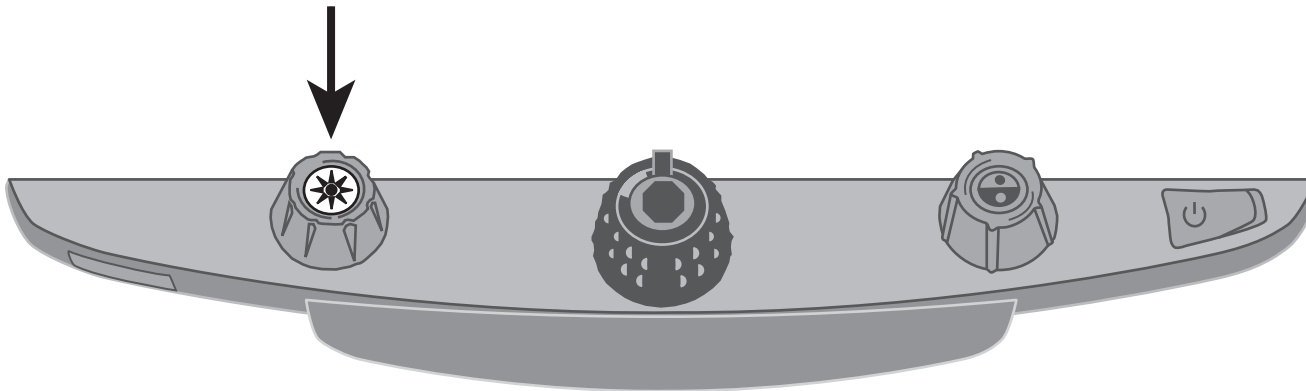11. Press and hold the find button located in the center of the yellow brightness dial to zoom out and see more of your document. A set of crosshairs is displayed on the monitor screen. Move the XY table and use the crosshairs to center and view a new area of the document. Release the find button to turn off the crosshairs and zoom back in on the new area of the document.


# Advanced Video Controls

## Focus lock and position locator beam

By default, the camera automatically focuses on the closest object in view. However, if you want it to focus on a specific object such as the document on the reading table, you can turn on the focus lock feature. This prevents the camera from accidentally focusing on your hand or any other item that moves in front of the camera. During normal operation, we recommend that you leave the focus lock feature off.

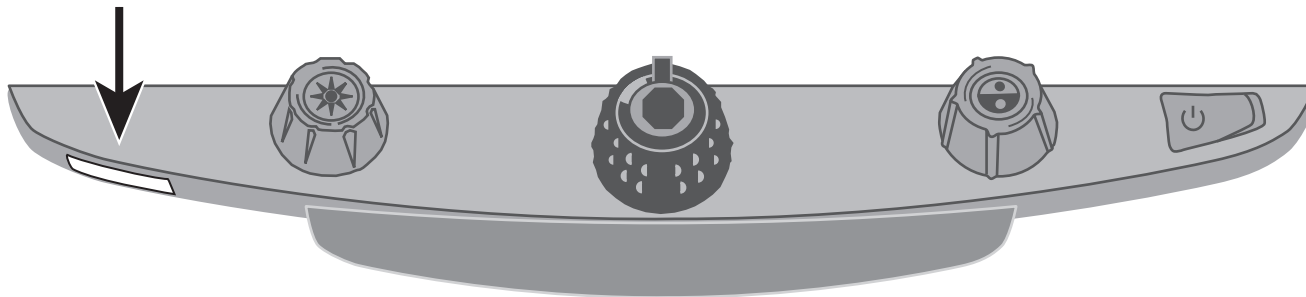1. Press the red focus lock button once to turn on the focus lock. The padlock icon appears on the screen: 
2. Press the button a second time to turn on the position locator beam and remain in focus lock mode.
3. Press it a third time to turn off both the beam and focus lock mode. Automatic focus is restored.


## Turning LED Reading Lights On or Off

Note: The **eMag 240 HD** has two reading lights designed to illuminate documents and other items you place on the reading table. This lighting is diffused to reduce glare and reflections. However, some items, such as photographs or glossy magazine pages, may be too shiny or reflective to view easily. When viewing such items, you can turn off the reading lights if you like.

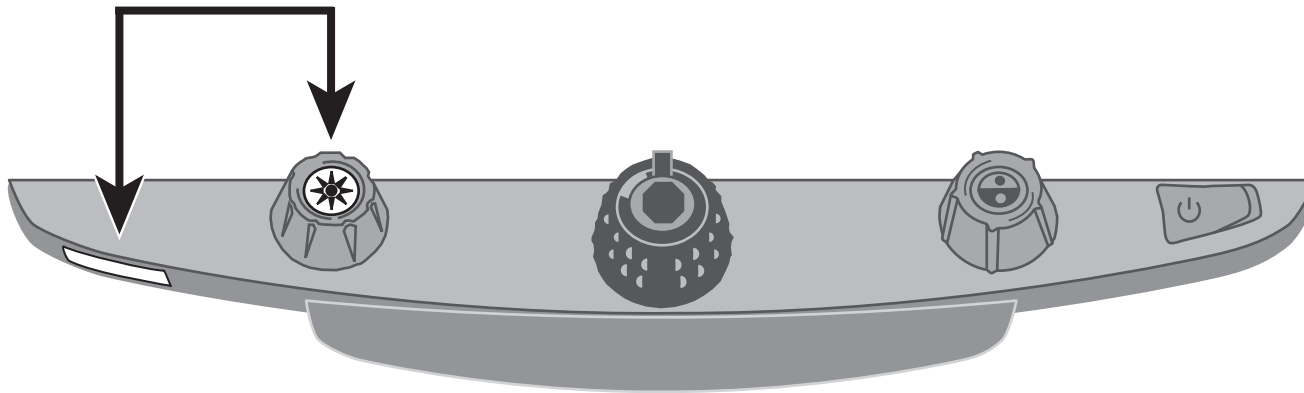
1. To turn off the LED reading lights, press the center of the yellow brightness dial and the red focus lock button simultaneously.
2. Press both again when you want to turn the reading lights on.

## Reading Lines and Reading Masks

Reading lines display as two horizontal or vertical lines on the screen. Reading masks block part of the screen to show only a horizontal or vertical slice of the screen. Use these items to help you keep your place while reading text. You can move either lines or masks together to change their position on the screen, or move one line or mask to adjust the distance between these items.

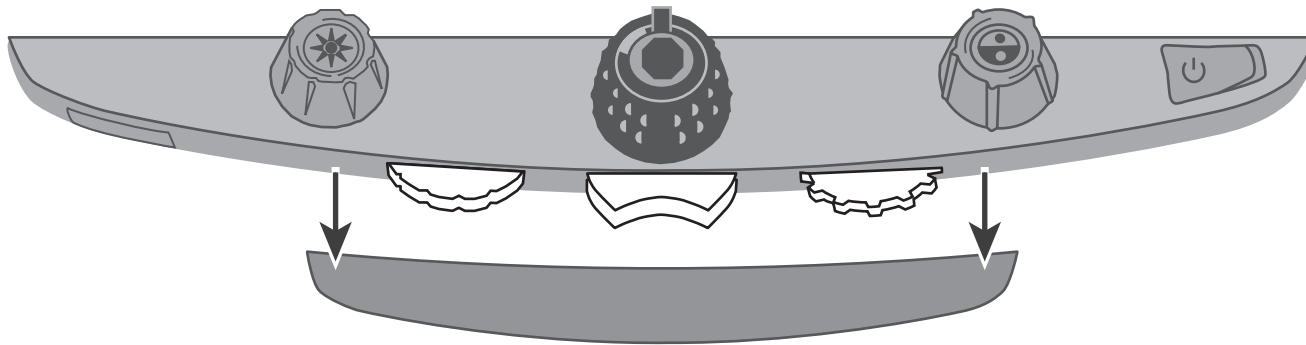Note: The buttons and dials for reading lines and reading masks are located under the plastic cover on the bottom of the control panel. Press the two recessed tabs on the back of the cover to remove it and access the purple switches and dials.


- Use the center purple switch to select horizontal or vertical reading lines and masks.
- Use the purple adjustment wheel on the left to move both lines and masks up and down or left and right.
- Use the purple width wheel on the right to change the spacing between reading lines and masks.

## Selecting Lines or Masks

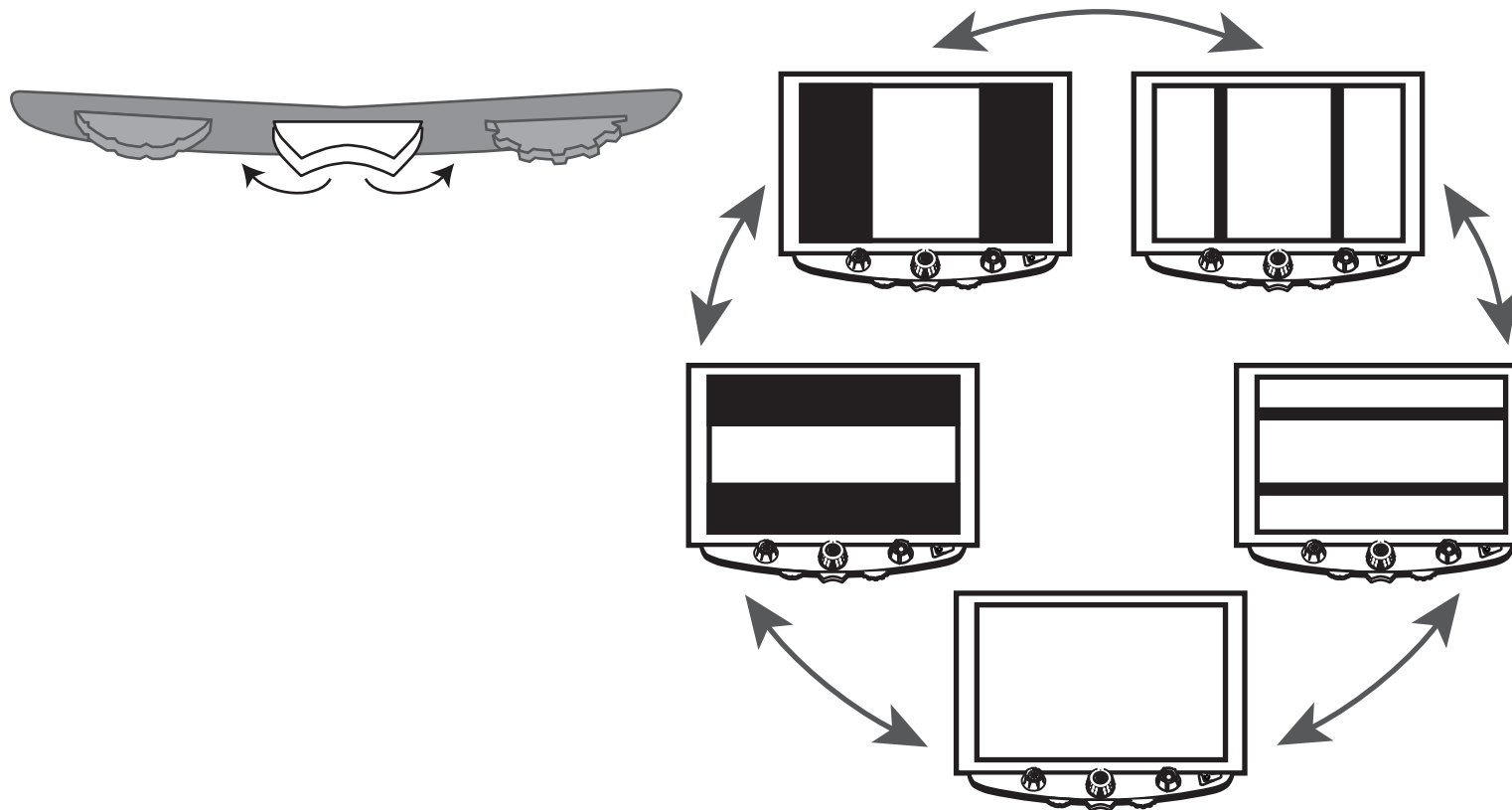
Use the center purple switch to display lines or masks on your screen. To do this:

- Press the center purple switch to the right to scroll through the following options: horizontal lines, vertical lines, horizontal masks and no lines/masks.
- Press the center purple switch to the left to scroll through the following options: horizontal lines, vertical lines, vertical masks, horizontal masks and no lines/masks.

## Opening Configuration Mode and Editing Settings

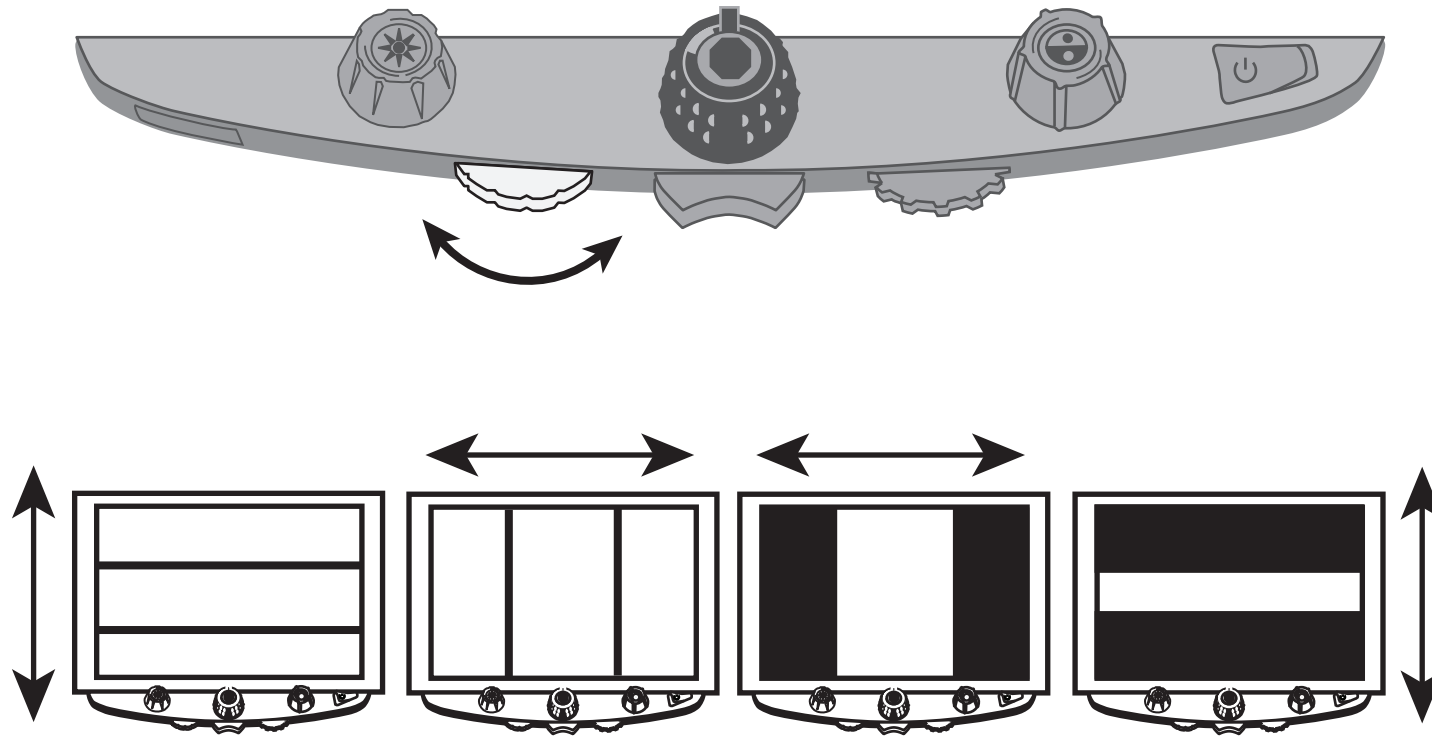- Turn the left purple adjustment wheel towards the right to move both lines and masks up and down or left and right on the screen.


## Adjusting Width between Lines or Masks

### For horizontal lines or masks:

- Turn the right purple width wheel to the left or right to move the top line or mask up or down. This increases or decreases the distance between horizontal lines or masks.

### For vertical lines or masks:

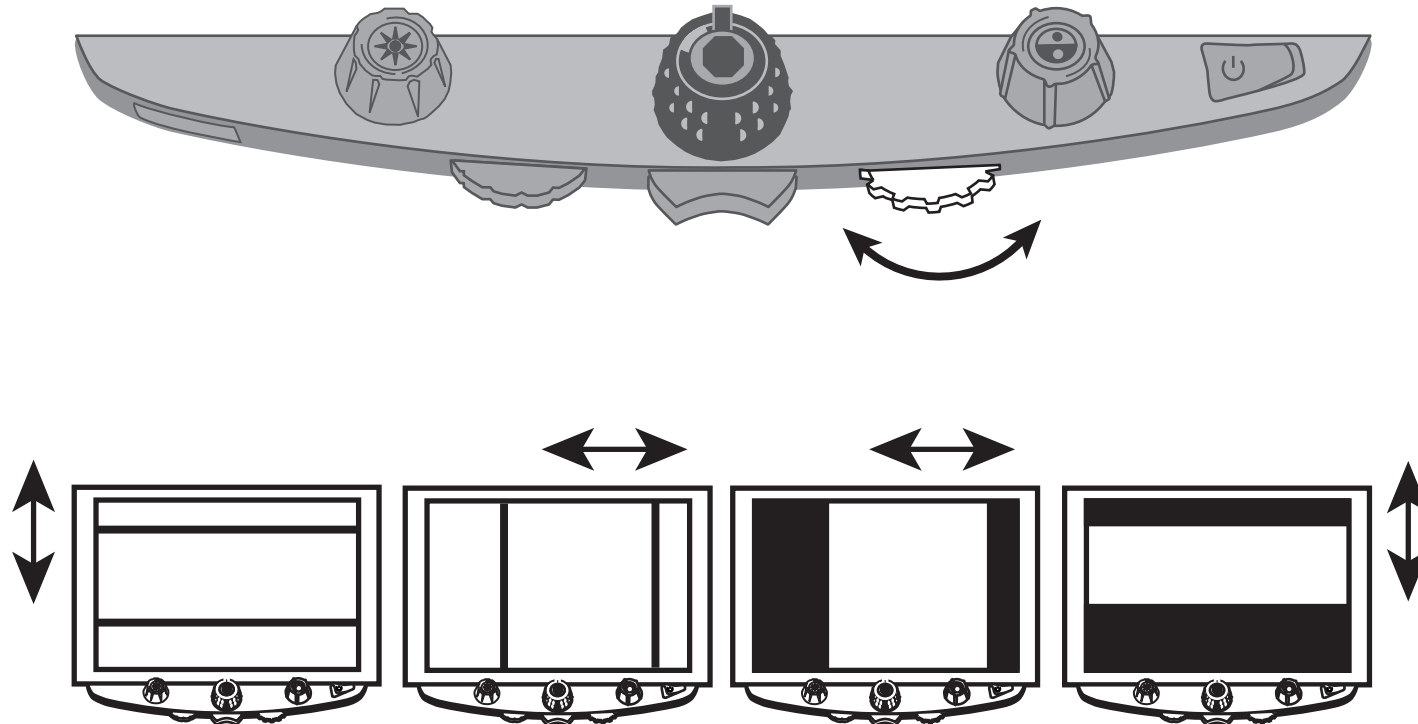- Turn the right purple width wheel to the left or right. This moves the right line or mask to the left or right. This increases or decreases the distance between vertical lines or masks.


## Using the eMag 240 HD Reading Line

The reading line is a single horizontal line that is shown on the screen to help you keep your place while reading a document. When using the reading table, it helps you to scroll your document across the screen and read it systematically.

To use the reading line, proceed as follows:

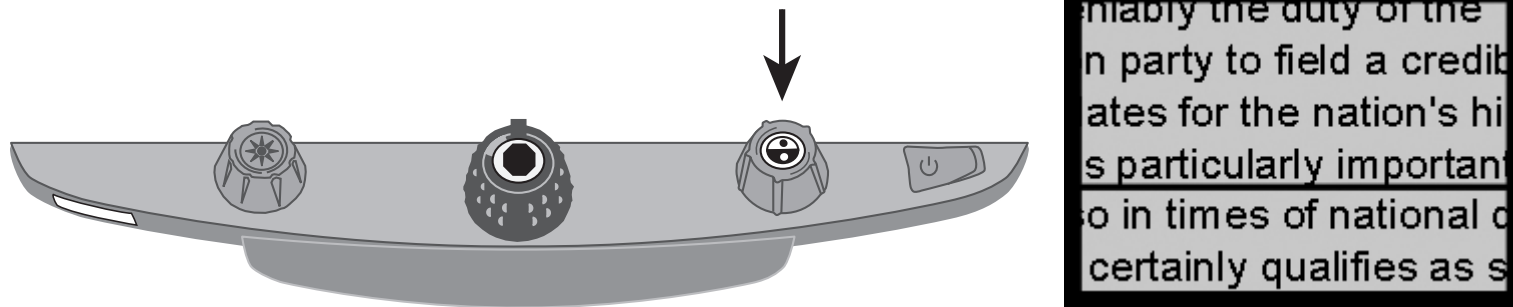1. Press the button in the center of the blue color select dial to place a horizontal line across the screen.

The line is displayed approximately one third of the way up from the bottom of the screen.

2. Next, move the reading table to the right to continue reading the text. To read the next line, move the reading table away from you so that the reading line is displayed under the next line of text. You can then slide the reading table to the left to continue reading.
3. To remove the reading line from the screen, press the blue color select dial again.

# Changing the Basic Configuration of the eMag 240 HD

## Settings

- The **eMag 240 HD** can be configured for up to 12 color combinations – saved as positions 1 through 12. Color combinations are used to show text for best readability, for example, yellow text against a blue background.
- Positions 1 through 3 are full color (true color), black on white and white on black. The color settings for these positions cannot be changed.
- Positions 4 through 12 can be customized or turned off. By default, positions 4 and 5 are yellow on blue and yellow on black, but they can be changed.

## Opening Configuration Mode and Editing Settings

1. When configuring color settings, it is best to have a page of text and a color photo to hand; position your document beneath the camera of the **eMag 240 HD**.
2. Press and hold the red focus lock button for approximately 10 seconds to enter the configuration menu. (Press and hold the button again to exit the configuration menu at any time and save your settings.)

Hollow outline:  
Colors cannot be changed

Solid:  
All settings can be edited

The letter T, which stands for Text Enhancement, appears in the lower left corner of the screen. Position 1 is always shown first. A hollow T is displayed for positions 1 through 3. This means that color settings cannot be changed; however, other settings for these positions can be edited. A solid T is displayed for positions that can be edited. The position number is also displayed on the screen: on the outside right.

3. Turn the blue color select dial to select the customizable positions 4 through 12.
4. Turn the yellow brightness dial to view all 33 available color settings. There are 30 two-color text enhancement modes, as well as true color (which is position 1), grayscale, and high color. When you have set the mode you wish to assign to this position, confirm (✓) by pressing the button in the center of the black magnification dial. An X icon (✗) means that the settings at this position are not saved.
5. Press the red focus lock button to switch to freeze frame mode: **||**. You can switch off the freeze frame function by pressing the button in the center of the black magnification dial: the check box becomes an X icon.
6. Press the red focus lock button to turn on the focus lock: **🔒**. You can switch off the focus lock function by pressing the button in the center of the black magnification dial: the check box becomes an X icon.
7. Press the red focus lock button again to jump to the magnification levels: **AAA**. The default setting is 1. Setting 2 gives you finer adjustment in the lower magnification range (positions 1 through 5); Setting 3 gives you finer adjustment in the medium magnification range (positions 6 through 10). Press the button in the center of the black magnification dial to select setting 1, 2, or 3.

8. Press the red focus lock button again to jump to the numbers for the magnification ranges: . Press the button in the center of the black magnification dial to select how the magnification level is displayed on the screen:

Position number (1 through 16)

No display

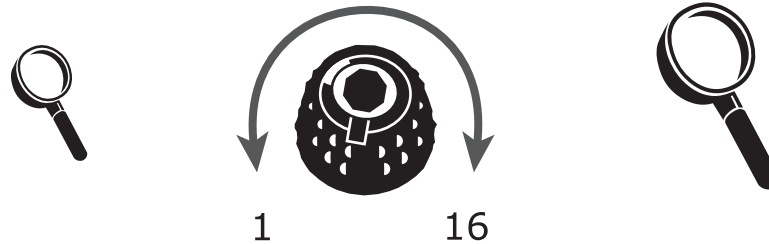
X value (magnification power – 3.5x, 6x, 10x and so on)

9. Press and hold the red focus lock button for 10 seconds to exit configuration mode and save the settings.

## Customizing Magnification Settings

The **eMag 240 HD** can display 16 levels of magnification. To increase magnification, turn the black magnification dial clockwise. To decrease magnification, turn it counterclockwise.

To allow you to get the most out of your **eMag 240 HD**, there are three magnification ranges from which you can choose the best magnification power for you.

- Range 1 is the default setting, and is ideal for most users. Turn the black magnification dial to move step by step through positions 1 through 16.
- Range 2 is ideal if you are comfortable with lower magnification. It provides finer adjustments between positions 1 through 8.
- Range 3 provides finer adjustments for midrange magnification available between positions 8 through 14. This range is good if you need greater magnification.

Experiment with the different magnification ranges and use the one that is best for you. You can always revert to range 1 at any time if you are not comfortable with the magnification available in ranges 2 and 3. The following describes how to select and edit your viewing preferences.

## Selecting a Magnification Range

1. Press and hold the red focus lock button for approximately 10 seconds until the letter T appears in the lower left corner of the screen.
2. Continue to press the red focus lock button until this icon is displayed on the screen.

3. Press the button in the center of the black magnification dial to select one of the three magnification ranges. For an overview of the magnification ranges, refer to the descriptions provided earlier in this guide.
4. Do one of the following:
  - To save the changes and exit configuration mode, press and hold the red focus lock button for approximately 10 seconds – until the icon at the bottom of the screen disappears.
  - To customize how magnification values are displayed on the screen, follow the steps described in the next section.

## Changing the Zoom Position Display

Note: The following assumes that you are proceeding from the magnification range. If not, press and hold the red focus lock button for approximately 10 seconds to open the **eMag 240 HD** settings. Then continue to press the red focus lock button until the magnifying glass icon appears on the screen.

Once you have selected a magnification range, you can choose whether or not the current magnification value is shown on the screen as you turn the black magnification dial. Proceed as follows to set your preference:

1. Press the red focus lock button until the magnifying glass icon is displayed on the screen.
2. Press the button in the center of the black magnification dial to select how magnification values are to be displayed via the magnifying glass icon.

Zoom Position Display  
Positions 1 through 16

Display of current  
magnification power  
(2.2x, 3.9x, 7.5x etc.)


Without display of  
the magnification power

Select one of the three options:

- Select the icon with the number 88 to display the magnification currently in use as one of the 16 positions on the black magnification dial.
  - Select the X icon to display the magnification currently in use with the magnification power – for example 1.9x, 3.4x, 8.4x and so on.
  - Select the icon with the clear magnifying glass if you do not want to see a magnification value display as you turn the black magnification dial.
3. To save changes and exit configuration mode, press and hold the red focus lock button for approximately 10 seconds until the icon at the bottom of the screen disappears.  
To discard any changes made while configuration mode is open, do not touch any control panel dials, buttons, or switches for approximately 30 seconds. This closes configuration mode without saving changes.

## Resetting Factory Default

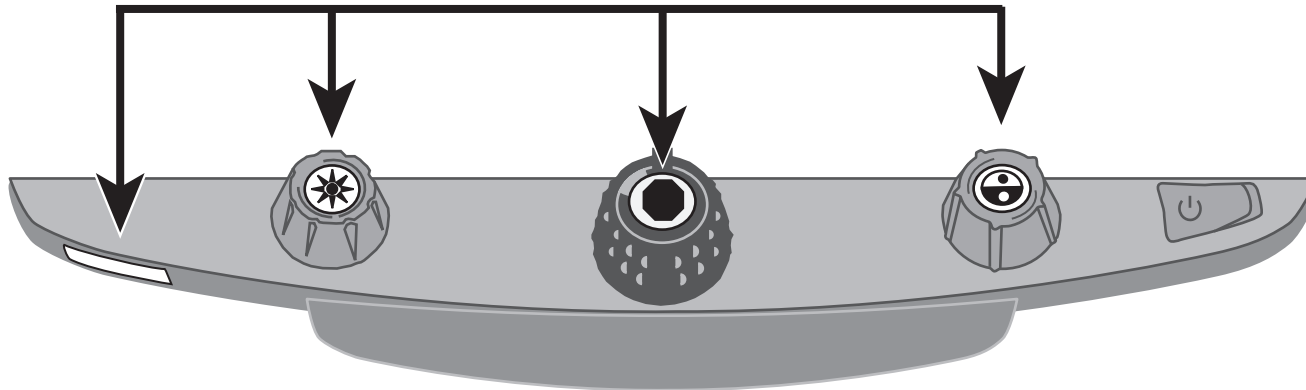
Press and hold the following dials and buttons simultaneously for approximately 10 seconds to reset the **eMag 240 HD** to its factory settings. Release the dials and buttons when the screen goes blank.

- the red focus lock button
- the button in the center of the yellow brightness dial
- the button in the center of the black magnification dial
- the button in the center of the blue color select dial

## Displaying Firmware Version Number

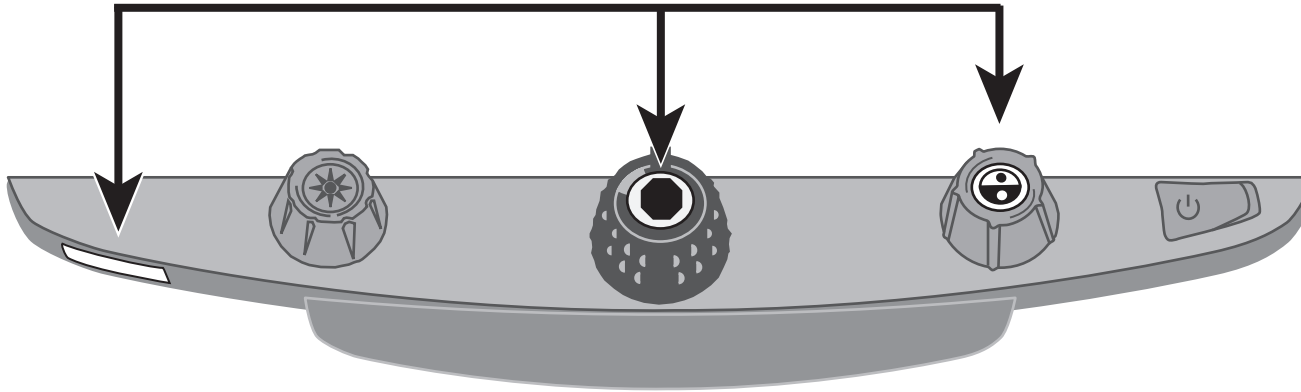
Press and hold the following dials and buttons simultaneously for 5 seconds to display the firmware version number. The word Ver. followed by three sets of numbers, which represent the major and minor firmware numbers, appear one after the other on the screen.

- the red focus lock button
- the button in the center of the black magnification dial
- the button in the center of the blue color select dial


# Connectors on the back of the unit

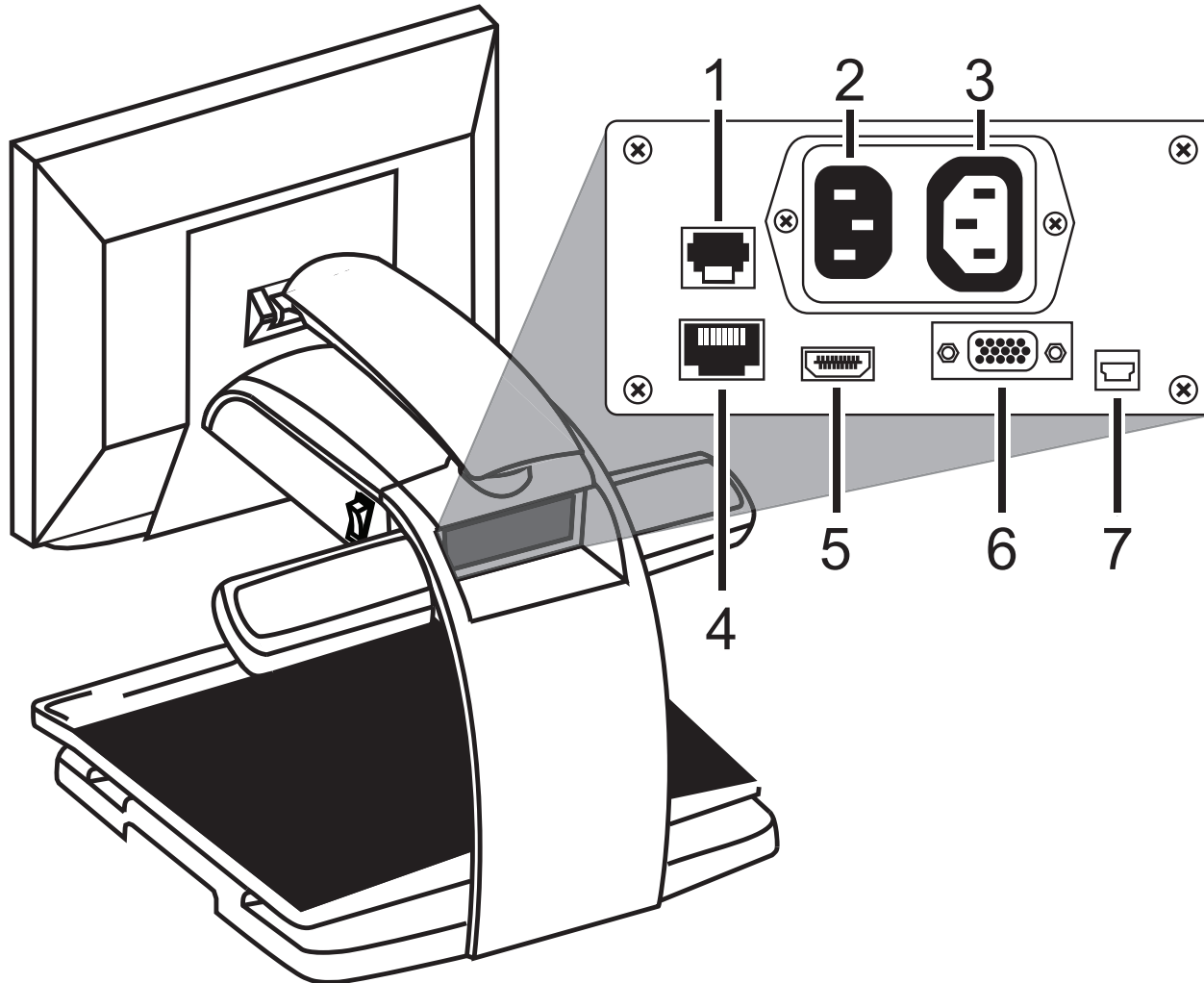Use this image and the descriptions on the next page to identify the **eMag 240 HD** connectors.

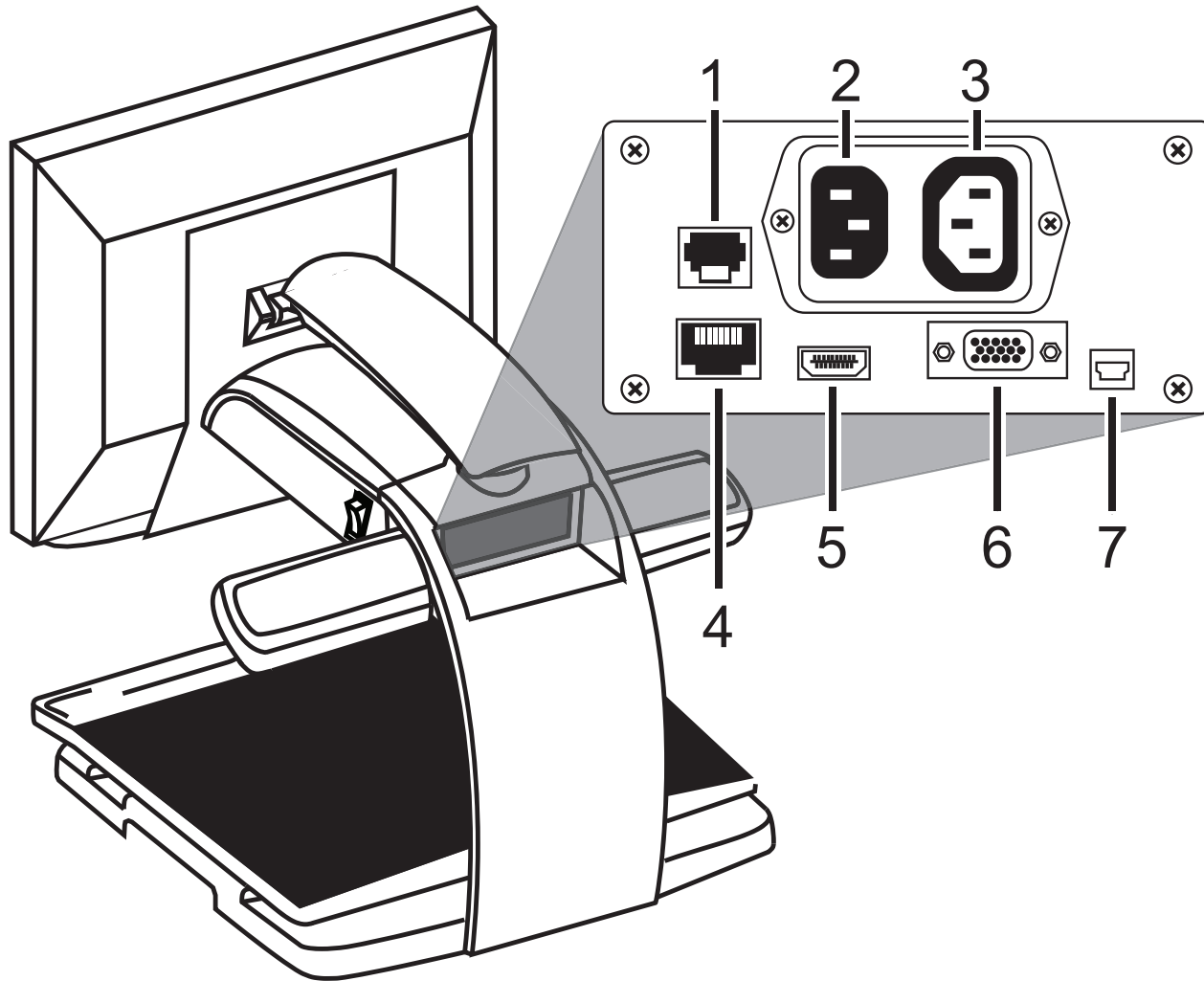1. Port for the control panel:  
Modular jack that connects the control panel with the **eMag 240 HD** camera. Do not disconnect the cable unless servicing the unit.
2. Power port: Plug the mains connector from the wall outlet into the **eMag 240 HD**.
3. Monitor power connector: Plug the mains connector from the monitor into the **eMag 240 HD**.
4. Test port: Do not plug in any cables here.

5. Monitor DVI port: Connects the **eMag 240 HD** monitor with the **eMag 240 HD** camera. There is no need to remove this monitor cable during normal operation.
6. VGA video port (15-pin)
7. USB connector

# Important Safety Information

Read and follow all warning notices and instructions marked on the product or included in the User's Guide.

- Slots and openings in the product are provided for ventilation. To ensure reliable operation of the product and to protect it from overheating, these slots and openings must not be blocked or covered.
- Do not allow anything to rest on the power cable, and do not set up the product where persons will step on the power cable.
- Do not attempt to repair the product yourself: Opening the unit or removing covers can expose you to dangerous voltages or other risks. Refer all servicing to qualified service personnel.
- When installed, the product must comply with the applicable Safety Standards and regulatory requirements of the country in which it is installed. If necessary, consult with the appropriate regulatory agencies and inspection authorities to ensure compliance.
- Always operate the product in a place where the temperature is between 10 °C and 35 °C (50° and 95° F).
- Keep it away from wet areas, rain, and other liquids. Do not submerge the product.
- Do not stare directly into the reading lamp, as the bright light can be harmful to the eyes.
- Do not attempt to move the product without first locking the monitor and reading table.

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Copyright © 2015 A. SCHWEIZER GmbH. Tous droits réservés.
Copyright © 2015 A. SCHWEIZER GmbH. Alle rechten voorbehouden.